



- Sublime & Impressions forms are 100% silicon which creates a natural friction, preventing shifting during wear. No more readjusting.
- Tapered edges create a secure seamless fit and reduced gaping.
- Lightweight honeycomb design allows for the form to be worn all day long without neck and shoulder strain.
- Sublime & Impressions forms will not leak.
- You can wear Sublime and Impressions forms directly against your skin for a most intimate fit, without the need for pocketed bras.


HOW TO INSERT THE FORM INTO A POCKETED BRA

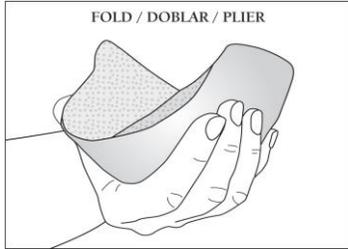


Gently cup the form in your palm and apply pressure to curl the form

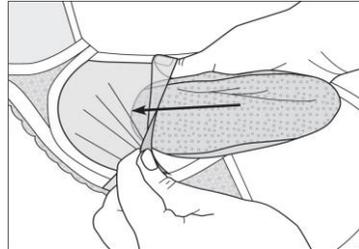


Guide the form with your hand into the pocket

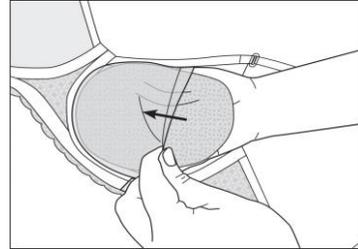

Use your hand to position the form

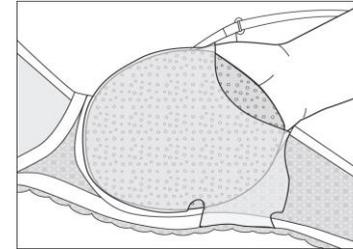


Gently remove your hand to leave the form in position

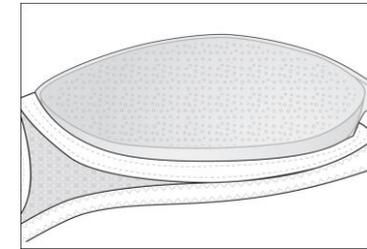

When placed in the pocket, the bottom rim of the form should sit slightly over the edge of the pocket allowing the form to rest flat against the chest wall.

DO NOT PULL On the Breast Form

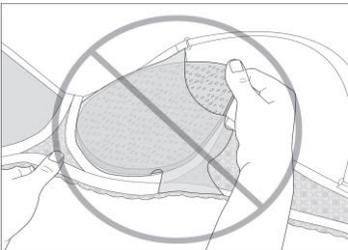


Always use your hand to guide the form in and out of the pocket


USING A COVER:

You may use a cover on your Sublime or Impressions Form. With the use of a cover the form will no longer have the natural friction, allowing the form to slide in the pocket with ease and may move within the pocket


EASY REMOVAL STEPS

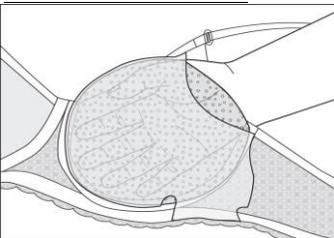

Use your hand as a guide to remove the form

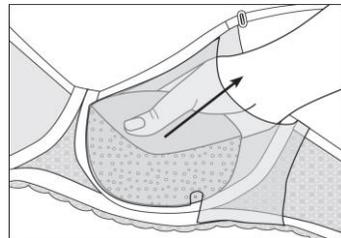

Gently curl the form with a small amount of pressure

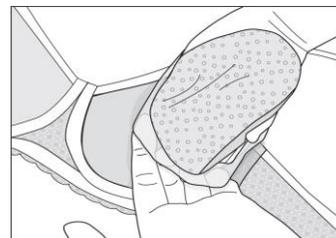

Gently guide the form out from the pocket

